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SL

Indications

The tube is inserted in case of need to relieve the stomach (e.g., during abdominal surgery), empty the stomach and feed the patient via the stomach tube.

Contraindications

The product should not be used on patients:

- With known hypersensitivity to any of the materials employed.
- They have severe facial injuries or a suspected skull base fracture.
 - In these cases, a duodenal tube can be inserted through the mouth.
- They have esophageal atresia or stricture.
- They consumed corrosive substances (acids, lyes).

Relative contraindications:

- Recent stomach, esophagus or abdominal surgery.
- Recent throat or nose surgery.
- Esophageal burns.
- Zenker's diverticulum.

Possible side effects

Using of the product in case of patient hypersensitivity to any of materials employed can cause allergic reaction. When using an inadequately sized gastric tube, damage to the mucous membrane can occur.

Materials used

The product is made from medical grade polyvinylchloride which does not contain phthalates (PVC – DEHP FREE).

Instructions for use

Before insertion, the doctor determines the depth of the probe by measuring the distance from the tip of the nose through the ear to the xiphoid process. The dimensions of the probe must be adapted to the dimensions of the patient.

- Prepare hands before use following the hygienic instructions.
- Visually inspect the product for any mechanical damage. Use it immediately.
- Peel the packaging by using aseptic technique. Discard it if the packaging is not intact.
- Moisten the tip of the stomach probe with a moisturizing agent.
- Slowly and with circular movements, we introduce the moistened stomach probe into the nostril.
- When the probe is in the pharyngeal space just before passing into the esophagus, stop the introduction and check the position of the probe.
 - If the probe twists/gathers in the throat, pull it out a little.
- The patient should tilt his head slightly forward and swallow the saliva.
 - While swallowing, the probe is pushed over the epiglottis. Suffocation and coughing are signs that the probe is in the trachea - we need to pull it out to the throat and repeat the introduction.
- If there are no problems, the probe is inserted up to the mark for the length to which the probe should be inserted in the patient. Attach the probe to the root of the nose with an adhesive patch.
- Regardless of whether the channels are freely open or closed with a stopper, before removing the syringe, always seal the probe with pean over the joint and vice versa: only when the syringe is placed on the probe, the pean is released.
- Spray the stomach tube with water (30-50 ml) and close it with a cap (or press it with a pean over the fold). The gastric tube must always be blocked to prevent movement of liquid through it (possibility of aspiration into the respiratory tract) and air leakage into it (inflating - flatulence).
- Before extraction, the gastric tube must be clean and empty.
- Take off protective gloves and disinfect your hands.

Precautions

The product is intended for single use only.The product should be used only by qualified medical staff. Attention, in case of re-use a risk of infection with diseases transmitted by blood or other body fluids, or occurrence of bacterial infection is present. Sterility can only be guaranteed for unopened and intact packaging units. The product should be connected only on syringes or administration lines with male catheter connector. Do not resterilize. Improper use can cause bacterial infection. After single use, dispose the product in accordance with applicable standards. Maximal duration of use is 7 days. Maximal duration of use in combination with fat food is 24 hours. In case of adverse events occurring during use of this medical device, inform TIK d.o.o. and National Competent Authority on the event. If the product is not used according to the instructions issued by TIK d.o.o., the company shall not be responsible for any injury or other damage that may arise.

Storage

Risk of occurrence or transfer of infection can also be increased by damaged device peck pack or other materials. Unsuitable temperature or moisture in the room can terminate product sterility and affect its functionality. The product should be stored in original packaging in a dry place in temperature range from 5 to 40°C. The product should be protected from extensive heat or direct sunlight. Sterility for properly stored product is assured for 5 years unless package is opened or damaged.

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